PHILIPS

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Engineer Name: YOJIRO MOTOYAMA

Device Status: Conditional Pass

Event Type: Predictive / Preventative Maintenance

Service Activity: Visit 6a

PM schedule for this site: PM Plan Template DigitalDiagnost 4.x- High- 20 visits / 120 months

EQUIPMENT DETAILS

Product Name: DigitalDiagnost 4 High Performance

Product Number: 712031

Serial Number: SN16000415

Installed Product: 70301047

UDI:

	Min	Max	UoM	Measurement	Result	Comments
--	-----	-----	-----	-------------	--------	----------

UBM Calculation

Calculation

Initial date of Tube statistic data log					Jan 17, 2017	
Initial exposure count large focus				0		
Initial exposure count small focus				0		
Initial exposure count vario focus				0		
Final date of Tube statistic data log					Jun 14, 2023	
Final exposure count large focus				40722		
Final exposure count small focus				101285		
Final exposure count vario focus				7		
Number of hours						
Total initial count				0		
Total final count				142014		
Exposure difference				142014		
Number of days				2339		
Exposure per year				22161		
Categories					High	

PM Start

PM Start

Performing a Visual Inspection					PASS	
--------------------------------	--	--	--	--	------	--

	Min	Max	UoM	Measurement	Result	Comments
PM Start						
PM Start						
Switching On the system					PASS	
Checking system log files (Checking M Cabinet log files can be skipped if generator errors are not found in system log files)					PASS	
Checking system functions					PASS	
Checking the thermal overload switch					PASS	
Safety, Display and Indicating systems					PASS	
System Maintenance record					PASS	
Functional Status					PASS	
PM Wall Stand VS						
Wall Stand VS (Question set 1)						
Fixations					PASS	
PM Ceiling Suspension						
Functional Status						
Pre checks					PASS	
Functional Status1						
a. Functional Test Long Movement					PASS	
b. Functional Test Lateral Movement					PASS	
c. Functional Test Vertical Movement					PASS	
d. Functional Test Tilt and Swing					PASS	
Test of Geometry						
Vertical test					PASS	
Horizontal test(if wall stand is installed)					PASS	
Covers						
Rear Cover of telescopic carriage					PASS	
Centre cover of telescopic carriage					PASS	
Tube cover					PASS	
Fixations						
Fixations					PASS	
CSM/CSM3 - Component Integrity						
Component integrity					PASS	
Cleaning brush					Pass	
Catches					Pass	

	Min	Max	UoM	Measurement	Result	Comments
PM Ceiling Suspension						
CSM/CSM3 - Component Integrity						
Alpha Drive (CSM3)					Pass	
Cable Roller					Pass	
Cable Carriages					Pass	
X ray tube and display installation					Pass	
Telescopic end covers					Pass	
CSM/CSM3						
Vertical Column					PASS	
Tracking Unit					PASS	
Wrap Up						
Wrap up					PASS	
PM Workstation						
AWS - Acquisition Work Spot						
Functional Status					PASS	
Philips Remote Service Functionality Checked					FAIL	
Philips Remote Service Functionality Failed				Other reasons		Hospital does not accept Remote
Cleaning AWS and Accessories					PASS	
Hard Disk Maintenance					PASS	
Service Partition Maintenance					PASS	
Patient Database Maintenance					PASS	
PM M-Cabinet						
M-Cabinet (Question set 1)						
Functional Status					PASS	
M-Cabinet (Question set 2)						
Switching off the system (skip if GEST errors like arcing, etc are not present)					PASS	
Turning off the mains power (skip if GEST errors like arcing, etc are not present)					PASS	
Removing the covers (skip if GEST errors like arcing, etc are not present)					PASS	
Cable connections (skip if GEST errors like arcing, etc are not present)					PASS	
M-Cabinet (Question set 3)						
Functional Status					PASS	

	Min	Max	UoM	Measurement	Result	Comments
PM M-Cabinet						
M-Cabinet (Question set 3)						
Tube calibration					PASS	
Backup of Generator data					PASS	
Final steps					PASS	
PM X-ray Tubes						
SRO/RO/SRM X-ray Tube Assemblies						
Preparatory Work					PASS	
Mechanical Status X-Ray Tube Assy.					PASS	
Status of Cables and Connectors					PASS	
Final Work					PASS	
PM Collimators NICOL/Galileo/XD						
Collimators NICOL/Galileo/XD (Question set 2)						
General Inspection					PASS	
Cleaning and Lubrication					PASS	
PM Accessories						
Accessories						
Accessories for Wall Stands					PASS	
Accessories for Table TH					PASS	
Accessories for Table TH-S					PASS	
Accessories for Stitching					PASS	
Additional Grids					PASS	
Additional Dedicated Accessories					PASS	
Stitching Support					PASS	
ES Wall Stand VS						
Protective Earth Wall Stand VS						
2) Column screw	300		mOhm	67	Pass	
3) Wall box cover screw	300		mOhm	90	Pass	
Touch Current Wallstand VS						
2) Column screw	20		μA	2.7	Pass	
3) Wall box cover screw	20		μA	2.8	Pass	
4) Rear cover screw wall stand display	20		μA	2.9	Pass	
5) Holder screw remote control station	20		μA	2.8	Pass	

	Min	Max	UoM	Measurement	Result	Comments
ES Bucky Units Wall Stands						
Protective Earth Bucky unit Wall Stand (Fixed Detector)						
2) Grid ejector		300	mOhm	88	Pass	
6) Rear cover screw		300	mOhm	81	Pass	
Touch Current Bucky unit Wall Stand (Fixed Detector)						
2) Grid ejector		20	µA	2.3	Pass	
6) Rear cover screw		20	µA	2.3	Pass	
ES Ceiling Suspension						
Protective Earth CS						
4) Union nut of the high voltage cable		300	mOhm	52	Pass	
6) Accessory rail screw		300	mOhm	66	Pass	
7) Rear screw of the tube control handle		300	mOhm	76	Pass	
Touch Current CS						
4) Union nut of the high voltage cable		20	µA	2.3	Pass	
5) Key switch		20	µA	2.3	Pass	
6) Accessory rail screw		20	µA	2.3	Pass	
7) Rear screw of the tube control handle		20	µA	2.2	Pass	
8) Clutch disk		20	µA	2.2	Pass	
ES M-Cabinet						
Protective Earth M-Cabinet CXA						
2/B) Front cover		300	mOhm	21	Pass	
3/C) Top cover		300	mOhm	30	Pass	
4/D) Rear cover		300	mOhm	22	Pass	
5/E) Left cover		300	mOhm	28	Pass	
6/F) Right cover		300	mOhm	27	Pass	
Touch Current M-Cabinet CXA						
2/B) Front cover		20	µA	1.5	Pass	
3/C) Top cover		20	µA	1.3	Pass	
4/D) Rear cover		20	µA	1.3	Pass	
5/E) Left cover		20	µA	1.4	Pass	
6/F) Right cover		20	µA	1.2	Pass	
ES Workstation						
Protective Earth Acquisition Workspot						
2) Screw (rear side of PC)		300	mOhm	80	Pass	

	Min	Max	UoM	Measurement	Result	Comments
--	-----	-----	-----	-------------	--------	----------

ES Wall Connection Box

Protective Earth Wall Connection Box

2) Upper Left Bezel	300	mOhm	31	Pass	
3) Lower Left Bezel	300	mOhm	29	Pass	
4) Upper Right Bezel	300	mOhm	33	Pass	
5) Lower Right Bezel	300	mOhm	21	Pass	
6) Front Cover Screw	300	mOhm	19	Pass	

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Number of Soft Failures

1

Philips Representative Signature:

Engineer Name: YOJIRO MOTOYAMA
 Date: Jun 14, 2023